

Endocube

organize - sterilize - pre-curve



What is EndoCube ?

The world's first chair-side GP organiser that allows you to intuitively place the GP cones in the same orientation as the tooth that you are working on.

It also lets you -

- Sterilize GP cones with Hypo
- Pre-curve your GP
- Stiffen GP before insertion
- Cut back your master cone precisely



Material and Clinical Usage

- 3D printed medical grade resin
- Auto-clavable for upto 1000 cycles
- Resistant to staining and Corrosion
- Resistant to breakage
- Resistant to hypochlorite and spirit



— How to pre-curve GP ?



- Gently insert GP
- Bend required angle and radius
- Hold bend for 10 seconds
- Optionally (for stiffness) add spirit and air dry, Voila !

— How to sterilise GP



- Channels are wide enough to bathe the GP with hypochlorite with a 2:1 volume ratio
- Fill with Hypo, Leave to sterilise for 60 seconds
- Upto 6 master cones simultaneously

Summary EndoCube

- 3D printed auto-clavable GP organiser
- Prevents your GP from ever getting mixed up
- 4 orientations -
 - Maxillary Molar - MB, MB2, DB, P
 - Mandibular Molar - MB, ML, DB, DL
 - 2 Premolars - Buccal and Palatal
 - Anterior 6
- Each channel designed to hold hypochlorite to sterilise master cones all at once
- Pre-curving tool to precisely bend GP to any configuration
- GP cutting tool (first of it's kind)
 - 0.5mm short
 - 1 mm short
 - 2 mm short
 - 3mm short



Design

Mandibular
Molars




2 Premolars

Anterior
Sextant



GP cutting
Tool

GP Pre-curling
Tool

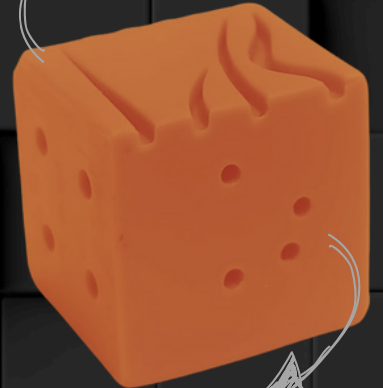

Maxillary
Molar

How to get yours

- Step 1 - Click this link
- Step 2 - Enter your Details
- Step 3 - Pay on QR Code
- Step 4 - Whatsapp 9810664561

Cost - Rs 2000
*including GST
**free shipping within India

